



		Dilution	Combook
Application	Frequency	Dilution per 10 litres	Contact Time
THEATRE AND STERILE PROCEDURES WALLS/TILES			
 Wash all surfaces from top to bottom using F10SCXD, rinse with water Wash off visible dirt (e.g. blood) using F10SCXD 	Weekly When visibly	40ml/10l water 40ml/10l water	
CHAIRS - Trigger spray using F10SC and wipe down with paper towel CUPBOARDS/SHELVES	soiled Weekly	40ml/10l	Leave to dry
 Wipe down (damp dust) using F10SC and clean cloth 	Weekly	40ml/10l water	Leave to dry
FLOORS - Sweep or rinse away all debris	Daily/after use		,
 Wash thoroughly using F10SCXD & broom or HP Washer, rinse with water Apply F10SC through fogging machine or garden sprayer Deep clean using F919SC – Apply product diluted with preferably warm 	Daily/after use After cleaning Monthly	40ml/10l water 40ml/10l water 200ml/10l water	Leave to dry 5 min warm water
water through HP washer, leave for 5 minutes, scrub lightly with broom and rinse with clean water. When using with cold water, allow contact time of 10 – 15 min before scrubbing			10 min cold water
FLOOR DRAINS	Weekly	200ml/10l	5-10 min
 Remove drain covers/traps and scrub with F919SC Flush with large volumes of clean water 	Weekly	water	
- Pour 1 litre of diluted F10SC down drain AIR SPACES	Weekly	40ml/10l water	
 Fog air spaces using F10SC through fogging machine or garden sprayer OR 	Weekly/After cleaning	40ml/10l water	Leave to dry
- Use F10 DISINFECTANT AEROSAL FOGGER OVERHEAD HOIST MECHANISM AND SHACKLES	cleaning	Ready to use	
 Wash thoroughly using F10SCXD and brush or HP Washer, rinse with 		40ml/10l water	
 water Apply F10SC through fogging machine or garden sprayer Deep clean using F919SC – Apply product diluted with preferably warm water through HP washer, leave for 5 minutes, scrub with broom and rinse with clean water. When using with cold water, allow contact time of 	Weekly Weekly Monthly	40ml/10l water 200ml/10l water	Leave to dry 5 min warm water 10 min cold
10 – 15 min before scrubbing			water
EQUIPMENT(O/H LIGHTS, Q-CARTS,SUCTION BOTTLES,FACE MASKS,ET TUBES,ANAESTHETIC EQUIPMENT,ULTRASOUND PROBES,ECG MACHINE AND CABLES,DRIP STANDS)			
 Wash/wipe all surfaces from top to bottom using F10SCXD, rinse with water 	Daily/Weekly/ After use	40ml/10l water 40ml/10l water	Leave to dry Leave to dry
 Wash bottles and tubing using F10SCXD, rinse with water Apply F10SC to all surfaces 	After use	40ml/10l water 40ml/10l water	Leave to dry
- Rinse bottles and tubing using F10SC OR	After cleaning After cleaning	Toma for water	
- Spray out tubing using F10 DISINFECTANT AEROSAL THERMOMETERS	After cleaning	Ready to use	Leave to dry
 Use two sets, one in use and one stored dry After use wipe with disposable paper 	After use		
- Wipe using F10 WIPES or spray with F10SC and wipe with paper towel SCOPES(ENDOSCOPES/GASTROSCOPES/ARTHROSCOPES)	After use	40ml/10l water	
 Follow manufacturers instructions with regards to rinsing procedure after use, as well as use of enzymatic agent 	After every use After every use	100ml/10l	
 Rinse for 1 minute. Wash using F10SCXD for 10 minutes if mechanical (5 minutes if ultrasonic), rinse for 1 minute 	After every use	water Ready to use	
 Soak in F10 STERILANT (RTU) for 10 minutes, rinse with sterile water or use without rinsing 	After every use	100ml/10l	
OR - Soak in F10SC (1:100) for 10 minutes, rinse with sterile water or use without rinsing		water	
NOTE: If a gluteraldehyde had been used to disinfect the instrument, it must be thoroughly washed and wiped with alcohol before immersed in			
F10. A reddish residue may form on the instrument as the result of the interaction between aldehyde and F10. This will not damage the Instrument and can be avoided by thorough cleansing to remove			
aldehyde residues			
Scrub with nylon brush using F10SCXD, rinse with water	After every use	40ml/10l water	





Application	Frequency	Dilution per 10 litres	Contact Time
- Trigger spray using F10SC	After cleaning	40ml10l water	Leave to dry
- Apply blade wash/lubricant	, and ordering		
- Store dry ENDOTRACHEAL TUBES			
- Wash thoroughly with F10SCXD (rinse lumen thoroughly to clear away	After every use	40ml/10l water	30 minutes
mucous)	After cleaning	40ml/10l water	10 minutes
Soak in F10SCOR soak in F10 STERILANT (RTU)	After cleaning	Ready to use	
 Rinse with sterile water OR remove without rinsing 			
- Store clean and dry NEEDLES,SYRINGES,I/V CATHETERS,DRIP SETS,SUTURE MATERIAL,NAIL			
BRUSHES			
 Single use is safest and best practice! Do not store in disinfectant 	After every use	100ml/10l	10 minutes
	Alter every use	water	10 minutes
 Wash using F10SCXD, rinse with water Soak in F10 STERILANT(RTU) 	After every use	Ready to use	10 minutes
- Rinse with sterile water OR Remove without rinsing			
- Store clean and dry			
INSTRUMENTS - Cover soiled instruments with sterile damp swab to avoid blood drying on	During		
them	procedure After procedure	100ml/10l	
 Wash thoroughly in lukewarm water and F10SCXD using a soft nylon brush 	Aiter procedure	water	
- Dry thoroughly and lubricate with instrument lubrication. Dry			
Sterilize through moist steam or cold sterilization process COLD STERILIZATION			
- Fully immerse clean and dry instruments in fresh solution of F10	After cleaning	Ready to use	10 minutes
STERILANT(RTU) - Remove using sterile instrument or wearing sterile gloves(rinse with	and when required		
sterile water OR use without rinsing)	roquirou		
- Use immediately or store in sterile container			
AIRCON SYSTEMS - Remove filters, apply F919SC, leave for 10 min, scrub lightly, rinse with	Monthly	200ml/10l	10 minutes
water		water	
 Immerse filter in F10SC F10SC can also be sprayed or fogged into air ducting 	Monthly	40ml/10l water	
WASTE BINS	Monthly	40ml/10l water	Leave to dry
 Remove plastic liner, tie closed and discard Wash down using F10SCXD, rinse with water 	Daily	40ml/10l water	
 Trigger spray with F10SC inside new liner as well as top of bin 	Daily	40ml/10l water	Leave to dry
TELEPHONES/LIGHT SWITCHES/DOOR HANDLES - Trigger spray using F10SC onto paper towel or clean cloth and wipe	Daily/weekly/	40ml/10l water	Leave to dry
down	when soiled		,
WORK SURFACES /THEATRE TABLE/ INSTRUMENT TROLLEYS/THEATRE			
TABLE MATTRESS) - Wash visible debris off with clean water	After use/Daily	40ml/10l water	Leave to dry
 Wipe down with F10SC in trigger spray bottle and clean cloth 	Weekly/when	40ml/10l water	,
 Wash all surfaces from top to bottom using F10SCXD, rinse with clean water 	soiled		
 Deep clean using F919SC – Apply product diluted with preferably warm 	Monthly	200ml/10l	5-10 minutes
water through HP washer, leave for 5 minutes, scrub lightly with broom and rinse with clean water. When using with cold water, allow contact		water	
time of 10 – 15 min before scrubbing			
SOAP DISPENSERS - Fill dispensers halfway. DO NOT TOP UP	Before filling	40ml/10l water	
- When bottle is empty, wash using hot water and F10SCXD	Daily/after use	40ml/10l water	Leave to dry
- Trigger spray nozzles using F10SC			
BASINS - Wash thoroughly with F10SCXD, rinse with water	Daily/after use	40ml/10l water	
- Trigger spray with F10SC	Daily/after use	40ml/10l water	Leave to dry
 HAND HYGIENE Sterile & invasive procedures Wash hands with F10 HAND SCRUB & luke warm water, while keeping 	Pofore	41	
hands higher than elbows& using friction for 2 minutes,rinse,close tap	Before procedure for 2	4ml	
with elbows,dry hands with sterile towel from fingers to elbows	minutes		





		Dilution per	Contact
Application	Frequency	Dilution per 10 litres	Time
General practice - Wash hands with F10 ANTISEPTIC LIQUID SOAP using friction for 30 seconds, rinse, dry hands with paper towel & use paper to close tap	Between cases/after contamination for 30 sec	4ml	
- Use F10 HAND GEL as alternative for decontamination of hands PATIENT SKIN PREPARATION	Between cases/after contamination. Leave to dry (30	1 ml	
Clip wide area neatly and as short as possible using clean no 40 blade Wash area thoroughly using F10 HANDSCRUB OR F10 ANTISEPTIC LIQUID SOAP and lukewarm water, dab away excess moisture. Repeat until no visible dirt is left on skin	sec) Before procedure Before procedure	4ml (for 30 sec)	
- Spray F10SKIN PREP (RTU) and wipe with clean swab. Repeat - Move patient onto operating table, spray surgical site with F10 SKIN PREP (RTU) PROTECTIVE CLOTHING (SCRUBS, THEATRE GOWNS, OVERALLS,	Before procedure Before procudure	Ready to use Ready to use	30 sec 30 sec
DRAPES) - Soak item in F10SCXD prior to washing	After use	40ml/10l water	30 minutes
 Wash in washing machine, hang in sun to dry or use tumble dryer To eliminate persistent odours, add approximately 100ml of F10 ODOUR ELIMINATOR to wash water 	Daily for a week or as required	Ready to use	
- Use brooms & brushes with nylon bristles & plastic backing where possible			
 Rinse clear of all debris Wash brooms, brushes, cloths thoroughly in F10SCXD, hang to dry (do not rinse) 	End of shift		
 Scrub buckets & mop trolleys with F10SCXD, rinse with water Start shift with clean and dry equipment 	End of shift End of shift	40ml/10l water	Leave to dry
X-RAY EQUIPMENT (LEAD GLOVES AND GOWNS) - Wash using F10SCXD if visibly soiled - Trigger spray using F10SC and wipe down with paper towel. INSECT CONTROL	If visibly soiled Daily	40ml/10l water 40ml/10l water	Leave to dry
 Spray walls and ceilings in areas where fly control is required lightly with F10 ENVIRONMENTAL SPRAY WITH INSECTICIDE, repeat daily for 3 days, thereafter every 10 – 14 days or as required 	Daily for 3 days Every 14 days As required	Ready to use	
STABLE AREA STABLES (WALKWAYS/PASSAGES)WALLS & DOORS			
Muck out and remove debris Wash all surfaces from top to bottom using F10SCXD, rinse with water	Daily Weekly/After discharge	40ml/10l water	
- Apply F10SC to all surfaces through fogging machine or garden sprayer FLOORS	After cleaning	40ml/10l water	Leave to dry
- Wash using F10SCXD with broom or HP Washer, rinse with water	Monthly/After discharge	40ml/10l water	
- Apply F10SC to all surfaces through fogging machine or garden sprayer SPRING CLEAN	After cleaning	40ml/10l water	Leave to dry
 Deep clean using F919SC – Apply product diluted with preferably warm water through HP washer, leave for 5 minutes, scrub lightly with broom and rinse with clean water. When using with cold water, allow contact time of 10 – 15 min before scrubbing 	With change of each season	200ml/10l water	5-10 minutes
 Apply F10SC to all surfaces through fogging machine or garden sprayer 	After deep cleaning	40ml/10l water	Leave to dry
 WATER & FOOD TROUGHS / BUCKETS Rinse thoroughly, apply F919SC, leave for 10 min, scrub lightly, rinse with water 	After discharge/Weekly	200ml/10l water	
 Apply F10SC to all surfaces through fogging machine or garden sprayer CRUSHES/PROCEDURE AREAS Remove debris 	After discharge/Weekly After use	40ml/10l water	Leave to dry
- Wash/Scrub using F10SCXD with broom or HP Washer, rinse with water	After use	40ml/10l water	

NOTE: FOR FULL INFORMATION ON PRODUCTS MENTIONED ABOVE REFER TO OUR PRODUCT INFORMATION LEAFLETS.





	Application	Frequency	Dilution per 10 litres	Contact Time
-	Apply F10SC to all surfaces through fogging machine or garden sprayer	After use / Weekly	40ml/10l water	Leave to dry
CLEANI	NG EQUIPMENT Use brooms & brushes with nylon bristles & plastic backing where possible	End of shift		
-	Rinse clear of all debris Wash brooms, brushes, cloths thoroughly in F10SCXD, hang to dry (do	End of shift	40ml/10l water	Hang to dry
-	not rinse) Scrub buckets & mop trolleys with F10SCXD, rinse with water Start shift with clean and dry equipment	Weekly	40ml/10l water	
DRAINS	Remove drain covers/traps & scrub with F919SC	Weekly	200ml/10l	
- - -	Rinse with water Flush with large volumes of clean water Pour 1 I of diluted F10SC down drain	Weekly	water 4ml/1l water	
EQUIPN EQUIPN	IENT (HALTERS, TACK, SADDLE CLOTHS.GROOMING IENT)			
-	Apply F10SC to all surfaces through trigger spray bottle Wash all surfaces from top to bottom using F10SCXD, rinse with water, hang out to sun dry	Between horses Weekly	40ml/10l water 40ml/10l water	Leave to dry Hand to dry
SEASOI	NAL INSECT CONTROL & HYGIENE Spray walls and ceilings in areas where fly control is required lightly with F10 ENVIRONMENTAL SPRAY WITH INSECTICIDE, repeat daily for 3 days, thereafter every 10 – 14 days or as required	Daily for 3 days Every 14 days As required	Ready to use	
	IYGIENE /VISIBLY DIRTY HANDS			
-	Wash hand with F10 ANTISEPTIC LIQUID SOAP using friction for 30 sec, rinse, dry thoroughly	When visibly dirty	4ml	30 sec
BETWE	EN HORSES Dispense F10 HAND GEL onto hands, rub over all surfaces, allow to air dry OR	Between handling horses, when hands not	1ml	Leave to dry
-	Spray hands with F10SC and leave to dry(use trigger spray bottle)	visibly soiled	4ml/1l water	
TRANSI				
AFTER - -	Wash all surfaces from top to bottom using F10SCXD, rinse with water Apply F10SC to all surfaces through fogging machine or garden sprayer	After each use After cleaning	40ml/10l water 40ml/10l water	Leave to dry
ISOLAT IN CASE	ION E OF INFECTIOUS DISEASES E.G. SALMONELLA, STRANGLES			
• HA\	/E DEDICATED STAFF ASSIGNED TO ISOLATED CASES			
• HA\	/E DEDICATED TACK & GROOMING EQUIPMENT			
• MA	KE USE OF FOOT BATHS			
• CO	RRECT HAND HYGIENE!			
	NOT USE HIGH PRESSURE WASHER			
	**Section 1: Section 1	Daily	80ml/10l water	
-	Wash all surfaces from top to bottom using F10SCXD, rinse with water. DO NOT use high pressure cleaner for infectious cases Apply F10SC to all surfaces through garden sprayer or fogging machine	After discharge/Weekly After cleaning	80ml/10l water 80ml/10l water	Leave to dry
-	Remove all bedding and place in plastic bags, tie closed Deep clean using F919SC – Apply product diluted with preferably warm water through HP washer, leave for 5 minutes, scrub lightly with broom and rinse with clean water. When using with cold water, allow contact time of 10 – 15 min before scrubbing	Once horse has been removed After discharge	200ml/10l water	





Application	Frequency	Dilution per 10 litres	Contact Time
 WATER & FOOD TROUGHS/BUCKETS Deep clean using F919SC – Apply product diluted with preferably warm water through HP washer, leave for 5 minutes, scrub lightly with broom and rinse with clean water. When using with cold water, allow contact time of 10 – 15 min before scrubbing 	After discharge/Weekly	200ml/10l water	5-10 minutes
 Scrub using F10SCXD and nylon brush, rinse with water Apply F10SC to all surfaces through fogging machine or garden sprayer 	After discharge/Weekly After discharge/Weekly	80ml/10l water	Leave to dry
FOOTBATHS - Scrub using F10SCXD, rinse with water - Fill with F10SC to level of at least 15cm EQUIPMENT HALTERS, TACK, SADDLE CLOTHS, GROOMING EQUIPMENT & CLEANING EQUIPMENT	Daily Replace daily or when visibly soiled	80ml/10l water 80ml/10l water	
Apply F10SC to all surfaces through trigger spray bottle Wash all surfaces from top to bottom using F10SCXD, rinse with water HAND HYGIENE	After use Weekly & after horse removed from isolation	80ml/10l water 80ml/10l water	Leave to dry
AFTER HANDLING HORSE, EQUIPMENT Wash hand with F10 HAND SCRUB using friction for 30 sec, rinse, dry thoroughly	After making contact and before handling other horses or their equipment.	4ml	30 sec
Dispense F10 HAND GEL onto hands, rub over all surfaces, allow to air dry OR Spray hands with F10SC and leave to dry(use trigger spray bottle) TRANSPORT AFTER USE	Between handling horses, when hands not visibly soiled	1ml 4ml/1l water	Leave to dry
 Wash all surfaces from top to bottom using F10SCXD, rinse with water Apply F10SC to all surfaces through fogging machine or garden sprayer 	After each use before transporting other horses.	80ml/10lwater 80ml/10lwater	Leave to dry

NOTE: FOR FULL INFORMATION ON PRODUCTS MENTIONED ABOVE REFER TO OUR PRODUCT INFORMATION LEAFLETS. \odot Health and Hygiene (Pty) Ltd Oct 2009 update